DCM067-5
Ed. 01/2015

SALIVA CONTROL

IVD

LOT

See external label

2°C - 8°C

$\Sigma = 3 \times 2 \text{ mL}$

REF DKO067

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REF DKO067

INTENDED USE

This product is intended for in vitro diagnostic use in the quality control of immunoassays using saliva samples

PRECAUTIONS AND WARNINGS

For in vitro diagnostic use only. Do not pipette by mouth. Exercise all normal precautions required for handling laboratory reagents.

Since no method can offer complete assurance of the absence of any infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and should be disposed of accordingly. Safety Data Sheets are available on request and on the website www.diametra.com.

STORAGE AND STABILITY

Opened: store at 2-8°C. The product is stable until the expiry date printed on the label. Dispense only the required amount of product necessary to perform the work. Do not return any unused material to the original vial.

Unopened: store at 2-8°C. Material is stable until the expiry date printed on the label.

PREPARATION FOR USE

Control for immunoassays is liquid.

1. Mix gently the content of the vial for 5 minutes. Avoid foaming. Do not shake.
2. Refrigerate (2-8°C) the unused material. Mix the contents thoroughly before reusing.

MATERIALS PROVIDED

Saliva control:

- DCE045/06701-0 1 vial x 2 mL (low level)
DCE045/06702-0 1 vial x 2 mL (medium level)
DCE045/06703-0 1 vial x 2 mL (high level)

EQUIPMENT REQUIRED BUT NOT PROVIDED

Volumetric pipette

VALUE ASSIGNMENT

Each lot of immunoassay controls is submitted to a number of reference laboratories and values are assigned from a consensus of results obtained by these laboratories. A mean value has been assigned to this lot of control using an ELISA method.

DiaMetra S.r.l. Headquarter: Via Calabria, 15, 20090 Segrate (MI) Italy. Tel. +39-02-2139184 – 02-26921595 – Fax 0039-02-2133354.
Manufacture: via Pozzuolo 14, 06038 Spello (PG) Italy. Tel. +39-0742-24851 Fax 0039-0742-316197 E-mail: info@diametra.com

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